



Companion Animal Skeletal/Tissue X-Ray Imaging

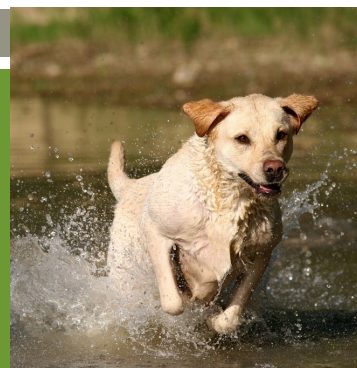

*Imaging like you have
never seen before!*

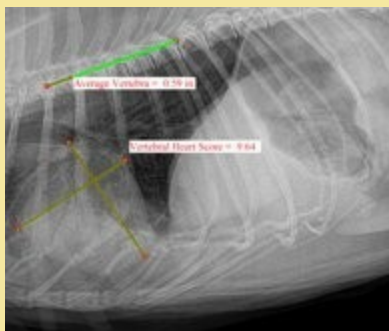

**METRON
SUPPORT SERVICES**

877-638-3868
metron@dvmconnexx.com



To the veterinarian, the perfect image is one that offers adequate detail to see pathology when pathology exists and be able to produce the same level of image detail every day.

Metron produces the perfect image by mastering the many factors that typically hinder image quality.



Go ahead! Zoom in to see how great imaging would benefit your diagnosis.

METRON
SUPPORT SERVICES

© 2009 - 2014 DVMCONNEXX. All rights reserved.

Companion Animal Skeletal/Tissue X-Ray Imaging

Technique and
Image Quality Standards




Metron uniquely incorporates the influences of CR Digital X-Ray to help produce that perfect image. In Companion Animal Skeletal and Tissue X-Ray Imaging, the image result is impacted by the following factors:


1. Portable X-Ray generator (ex. kVp80 MA15)
2. Tabletop X-Ray generator (ex. kVp120 MA400)
3. Distance from plane of interest
4. Thickness and density of the object of interest
5. X-Ray plate type and current condition
6. X-Ray technique
7. Custom X-Ray scan settings including resolution and filtering
8. Image quality management

With most CR digital X-Ray systems, the production of “perfect” is not possible. Metron changes all of that. Rather than ignore and blame, we incorporate the many factors that go into producing “perfect” images. Though “perfect” may be impossible, it certainly needs to be a goal. As the practitioner, you need to trust your tools to help you diagnose. The inventory of X-Ray images that follow have been produced by veterinarians throughout the world using Metron and our unique approach to “image quality management”.

These images have been produced in production and have not been modified from their original form.

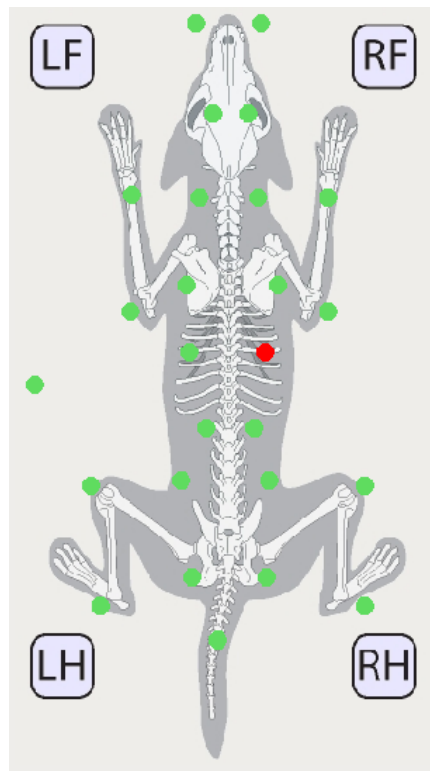

Canine Skeletal Chart

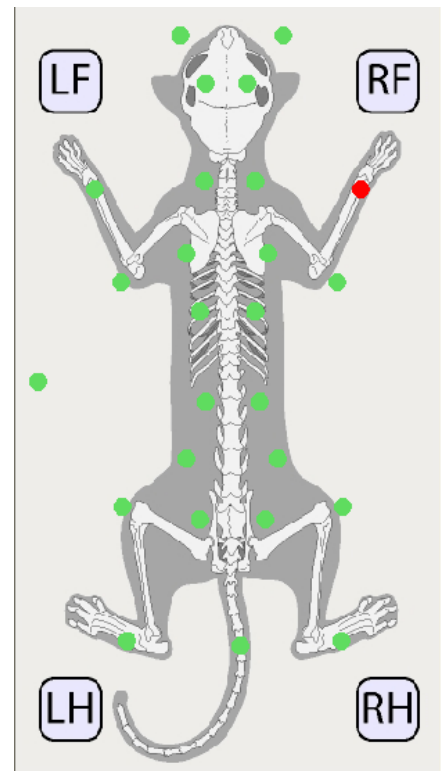

Feline Skeletal Chart

877-638-3868
metron@dvmconnexx.com

METRON
SUPPORT SERVICES

Companion Animal Skeletal/Tissue X-Ray Imaging

Technique and
Image Quality Standards



Go ahead! Zoom in to see how
great imaging would benefit
your diagnosis.



R Thorax DV

300MA X-Ray Generator
Canine 12cm R Thorax DV
Tabletop 65kVp 12MAS
Fuji plate, AllPro ScanX 12
Portable

Chanda Holschbach
Hospital Administrator
Packerland Veterinary Center
Green Bay, WI



R Thorax Lateral

400MA X-Ray Generator
Canine 16cm R Thorax Lateral
Bucky 72kVp 20MAS
Fuji plate, AllPro ScanX 14

Dr. Kate Kewish
Owner
Ellerslie Pet Hospital
Edmonton, AB, Canada



R Thorax Lateral

300MA X-Ray Generator
Feline 8cm R Thorax Lateral
Tabletop 54kVp 10MAS
Fuji plate, AllPro ScanX Trek

Clifford Park Veterinary Surgery
Toowoomba, Queensland,
Australia

877-638-3868

metron@dvmconnexx.com

METRON
SUPPORT SERVICES

Companion Animal Skeletal/Tissue X-Ray Imaging

Technique and
Image Quality Standards



Go ahead! Zoom in to see how
great imaging would benefit
your diagnosis.



R Stifle Lateral

400MA X-Ray Generator
Canine 4cm R Stifle Lateral
Tabletop 55kVp 5MAS
Fuji plate, AllPro ScanX 14

Dr. Kate Kewish

Owner

Ellerslie Pet Hospital
Edmonton, AB, Canada



RH Stifle Lateral Post Op

300MA X-Ray Generator
Canine 4cm
RH Stifle Lateral Post Op
Tabletop 55kVp 5MAS
Fuji plate, AllPro ScanX 12
Portable

Chanda Holschbach

Hospital Administrator

Packerland Veterinary Center
Green Bay, WI


877-638-3868

metron@dvmconnexx.com

METRON
SUPPORT SERVICES

Companion Animal Skeletal/Tissue X-Ray Imaging

Technique and
Image Quality Standards

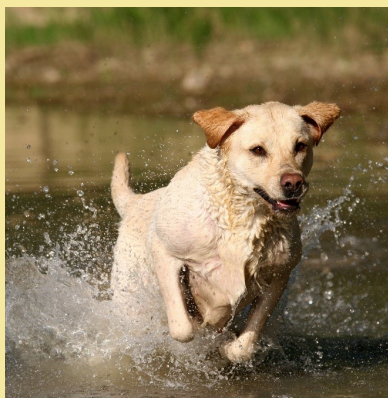

Go ahead! Zoom in to see how
great imaging would benefit
your diagnosis.



L Extraoral Lateral Oblique

400MA X-Ray Generator
Canine 7cm
L Extraoral Lateral Oblique
Tabletop 63kVp 10MAS
Fuji plate, AllPro ScanX 14


Dr. Kate Kewish
Owner
Ellerslie Pet Hospital
Edmonton, AB, Canada



LH Pelvis Lateral

400MA X-Ray Generator
Canine 12 cm LH Pelvis Lateral
Bucky 68kVp 18MAS
Fuji plate, AllPro ScanX 14

Dr. Kate Kewish
Owner
Ellerslie Pet Hospital
Edmonton, AB, Canada

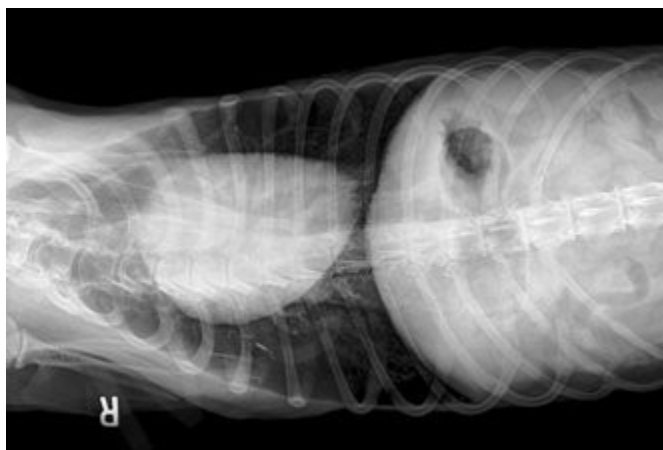

R Abdomen VD

400MA X-Ray Generator
Canine 23 cm R Abdomen VD
Bucky 83kVp 20MAS
Fuji plate, AllPro ScanX 14

Dr. Kate Kewish
Ellerslie Pet Hospital
Edmonton, AB, Canada

877-638-3868
metron@dvmconnexx.com

METRON
SUPPORT SERVICES